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## General information – ActiMaris® Forte wound irrigation solution (300ml)

### 1. Product profile

ActiMaris® Forte wound irrigation solution is made of water with sea salt (3%), NaOCl (0.2%) and active oxygen (O<sub>2</sub>). ActiMaris® Forte wound irrigation solution in interaction with the mucous membrane, skin and wound acts nascent. Active oxygen, sodium hypochlorite and sea salt have an eliminating effect on typical sore pathogens without damaging the tissue. The decontaminating effect against MRSA / VRE tested positive. The oxidative effect of a singlet oxygen (O<sub>2</sub>) and hypertonic saline are also responsible for the neutralization of wound odors. The alkaline pH of ActiMaris® Forte wound irrigation solution supported the provision of OH<sup>-</sup> ions (as part of a naturally balanced redox system) in addition, controlled the release of oxygen. Sea salt contains sodium chloride as its main ingredient (NaCl = 98%) as well as 2% other sea salts. The increased salinity of ActiMaris® Forte wound irrigation solution in turn ensures optimum hyperosmotic effect so that peri-wound swelling subsides quickly.

### 2. Fields of application

ActiMaris® Forte wound irrigation solution is suitable:  
For cleaning, humidification, decontamination, biofilm resolution, swelling and physiological debridement, to create an optimal „micro-environment“ for:

- acute mechanical wounds (skin lacerations, bites, cuts and abrasions, crack and contused) and postoperative wounds
- chronic wounds (pressure ulcers, arteriovenous ulcers, diabetic ulcers)
- neurotic, malodorous wounds and ulcerating tumors, even with cavities (Hollow spaces)
- thermal and chemical wounds (burns 1st to 3rd degree)
- entry portals of urological catheters and PEG tubes and drains
- venous ulcers

For the intra-operative cleaning and irrigation of superficial wounds.  
Moistening of bandages and wound dressings.

During dressing changes, loosen encrusted bandages or other encrusted wound dressings.  
For the local treatment of the skin and mucosa in inflammatory and infectious processes.  
For the prevention of infection of the mucosa and skin, burns and other wound types.

### 3. Instructions for use

ActiMaris® Forte wound irrigation solution can be applied at room temperature. In the event of strong pain, it is suggested that the necessary amount be heated to body temperature in a separate bowl before use (irrigation solution is to be heated to approximately 37°C).

Before using ActiMaris® Forte wound irrigation solution, the wounds should first be rinsed and cleaned with ActiMaris® Sensitive wound irrigation solution, in order to remove dirt as well as easily removable coatings. The subsequent deep cleansing and odor reduction will be implemented with ActiMaris® Forte wound irrigation solution; the soaked sterile bandages or wound dressings are applied for this purpose (see also section 9: Interactions). The application should preferably be in the form of the classic wet / dry phase („moist balance“).

**The recommended application time of the wet phase in case of severe fibrinous or purulent, infected or strong foul-smelling wounds is → 5-10 minutes.**

The duration and frequency of application should be adapted to the individual wound conditions. The use of the wound irrigation solution is not limited to time and frequency. Necrosis and coatings can be removed mechanically, depending on the wound condition. Primary surgical sanitation is advised with thick, adherent, fibrous or necrotic wound coatings.

After the improvement of the wound condition, it is advised to continue further wound cleansing with ActiMaris® Sensitive wound irrigation solution until the wound is visually clean.

ActiMaris® Sensitive wound irrigation solution is recommended for patients with sensitive skin.

Be careful when applying the wound irrigation solution so that the bottle does not touch the skin or wound of the patient in order to prevent contamination or the spread of micro-organisms.

### 4. Tissue tolerance and biocompatibility

ActiMaris® Forte wound irrigation solution is suitable for long-term (> 24h) as well as repeated use during dressing changes even with irritated skin and mucus. The application is painless and is generally well tolerated by allergy sufferers. Inhibition of granulation and epithelialization is not to be feared. ActiMaris® Forte wound irrigation solution is dermatologically assessed as safe.

### 5. Risks followed by improper use

- Improper use may cause softened wound edges
- Cross-contamination between wounds or patients with incorrect handling of the tube / bottle

### 6. Side effects

So far, in several years of clinical experience, the following symptoms occasionally occurred:

- Slight temporary burning sensation
- Slight burning and sneezing if applied to nose area,

### 7. Contraindications

Since allergies can never be excluded ActiMaris® irrigation solutions or ActiMaris® wound gel should not be used with known or suspected allergies to any of its ingredients.

### 8. Restrictions on use

#### Pregnancy and lactation:

There is no found evidence of genotoxicity, carcinogenicity or embryotoxic effects of the ingredients. Due to the lack of studies and long-term clinical experience with pregnant and breast feeding women, ActiMaris® Forte wound irrigation solution should be used in these cases only after an accurate medical assessment.

#### Infants and newborns:

Due to insufficient experience of long-term exposure, ActiMaris® Forte wound irrigation solution should only be applied after careful medical benefit-risk assessment.

### 9. Interactions

ActiMaris® Forte wound irrigation solution acts as a bio-physical compound due to the content of ROS substances (O<sub>2</sub> + OCl<sup>-</sup>) energy and oxidizing, which is why a simultaneous combination with other local, active substances and / or interactive wound dressings is in principle unnecessary. Contact with non-precious metals should be avoided. The wound can be covered with modern wound dressings after applying ActiMaris® Forte wound irrigation solution.

On the other hand, clinical trials have shown a good compatibility with modern wound dressings such as silver dressings. Likewise, there is ample clinical evidence of good tolerability when combined with inert, non-active bandages and wound dressings such as compresses, gauze, pads, sponges, gels, hydro fibers, alginates, hydrocolloids etc.

The concomitant use of ActiMaris® Forte wound irrigation solution with the ActiMaris® wound gel is possible and useful.

### 10. General safety instructions

ActiMaris® Forte wound irrigation solution is suitable only for external wounds on skin or mucosa. Do not use for infusion or injection! Do not swallow!  
Use only closed and undamaged containers. Keep out of reach of children!  
It is possible for colored fabrics to fade after contact with ActiMaris® products.

### 11. Composition and technical information

Sea salt, water, sodium oxychlorite.  
Clear, colorless, alkaline and hypertonic solution with characteristic smell; contains 3% sea salt (Sal Maris) and 0.2% NaOCl.

### 12. Storage specification, duration, expiry date

ActiMaris® Forte wound irrigation solution has to be stored at room temperature between +5° and +25°C. Protect from direct sunlight. Shelf life is 30 months. Expiry date is on the packaging. After opening the tube, the grace period is 3 months; always close tube tightly after use.

#### ActiMaris AG Manufacturer

ActiMaris AG  
Sandgrube 29  
9050 Appenzell  
info@actimaris.com  
Tel.: +41 71 505 75 25

LOT Lot number  
REF Order number  
Follow instructions for use  
Use by: year and month  
Storage (temperature)  
Protect from direct sunlight  
Manufacturer

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**ActiMaris AG**  **Fabrigant:**