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Stand der Information: 12/2018 - V6.1

Medizinprodukt: Klasse II b, CE 1250

## General information – ActiMaris® Sensitiv wound irrigation solution (300ml)

### 1. Product profile

ActiMaris® Sensitiv wound irrigation solution is made of water with sea salt (1,2%), NaOCl (0.04%), active oxygen (O<sub>2</sub>) and HOCl (0,0044%). ActiMaris® Sensitiv wound irrigation solution in interaction with the mucous membrane, skin and wound acts nascent. Active oxygen, sodium hypochlorite and sea salt have an eliminating effect on typical sore pathogens without damaging the tissue. The decontaminating effect against MRSA / VRE tested positive. The oxidative effect of singlet oxygen (O<sub>2</sub>) and hypertonic saline are also responsible for the neutralization of wound odors.

The alkaline pH-value of ActiMaris® Sensitiv wound irrigation solution is supported by the provision of OH<sup>-</sup> ions (as part of a naturally balanced redox system) in addition to the controlled release of oxygen.

Sea salt contains sodium chloride as its main ingredient (NaCl = 98%) as well as 2% other sea salts. The increased salinity of ActiMaris® Sensitiv wound irrigation solution in turn ensures optimum hyperosmotic effect so that peri-wound swelling subsides quickly.

### 2. Fields of application

ActiMaris® Sensitiv wound irrigation solution is suitable:

For cleaning, humidification, decontamination, biofilm resolution, swelling and physiological debridement, to create an optimal „micro-environment“ of:

- acute mechanical wounds (skin lacerations, bites, cuts and abrasions, crack and contused) and post-operative wounds
- chronic wounds (pressure ulcers, arteriovenous ulcers, diabetic ulcers)
- necrotic, malodorous wounds and ulcerating tumors, even with cavities (Hollow spaces)
- thermal and chemical wounds (burns 1st to 3rd degree)
- entry portals of urological catheters and PEG tubes and drains
- venous ulcers
- intrathoracic, peritoneal and ostomy rinses

For the intra-operative cleaning and irrigation of superficial wounds.

Moistening of bandages and wound dressings.

During dressing changes to loosen encrusted bandages or other encrusted wound dressings.

For the local treatment of the skin and mucosa in inflammatory and infectious processes.

For the prevention of infection at the mucosa and skin, burns and other wound types.

### 3. Instructions for use

ActiMaris® Sensitiv wound irrigation solution can be applied at room temperature. In the event of strong pain, it is suggested that the necessary amount be heated to body temperature in a separate bowl before use (irrigation solution is to be heated to approximately 37°C). The duration and frequency of application should be adapted to the individual wound conditions and should be carried out until all deposits and necrosis are easily removed and the wound is visually clean. Depending on the wound condition, a bandage change can be done according to indications and phases, every 2 to 3 days. In case of non-infected wounds, necrosis and coatings can be mechanically removed depending on the wound condition. In cases of difficult to remove bandages it is recommended to hydrate the wound dressings with wound irrigation solution ActiMaris® (Forte or Sensitive) for at least 5 minutes until a smooth removal of bandages without trauma to the wound surface is possible. A primary surgical therapy is recommended with thick, firmly adhering fibrinous or necrotic wound deposits.

For cleaning wounds, ActiMaris® Sensitiv wound irrigation solution soaked bandages (see also section 9: Interactions) can be combined with inert dressings or other wound dressings.

**The application should preferably be in the form of the classic wet / dry phase («moist balance»).** The duration of the wet phase is to be adjusted accordingly depending on the thickness and strength of the fibrin, microbial colonization, wound edge maceration or irritation.

- Stable wound environment, clean wound bed without microbial presence:
  - Recommendation: 5-10 minutes of application time
- Fibrin coating, without microbial presence, also applied to wound edge maceration or irritation:
  - Recommendation: 15 minutes of application time
- Fibrin coating, foul-smelling wounds, heavily populated by bacteria:
  - Recommendation: 20 minutes of application time

Be careful when applying the wound irrigation solution so that the bottle does not touch the skin or wound of the patient in order to prevent contamination or the spread of micro-organisms.

### 4. Tissue tolerance and biocompatibility

ActiMaris® Sensitiv wound irrigation solution is suitable for long-term use even on sensitive / irritated skin and mucosa due to its non-irritating and well-tolerated application. There is no inhibition of granulation and epithelialisation. Dermatologically approved.

### 5. Risks followed by improper use

- Improper use may cause softened wound edges
- Cross-contamination between wounds or patients with incorrect handling of the tube / bottle

### 6. Side effects

So far, in several years of clinical experience, the following symptoms occasionally occurred

- Slight temporary burning sensation
- Slight burning and sneezing if applied to nose area,

### 7. Contraindications

Since allergies can never be excluded ActiMaris® irrigation solutions or ActiMaris® wound gel should not be used with known or suspected allergies to any of its ingredients.

### 8. Restrictions on use

#### Pregnancy and lactation:

There is no found evidence of genotoxicity, carcinogenicity or embryotoxic effects of the ingredients. Due to the lack of studies and long-term clinical experience with pregnant and breast feeding women, ActiMaris® Sensitiv wound irrigation solution should be used in these cases only after an accurate medical assessment.

#### Infants and newborns:

Due to insufficient experience of long-term exposure, ActiMaris® Sensitiv wound irrigation solution should only be applied after careful medical benefit-risk assessment.

### 9. Interactions

ActiMaris® Sensitiv wound irrigation solution acts as a bio-physical compound due to the content of ROS substances (O<sub>2</sub> + OCl<sup>-</sup>), energy and oxidation, which is why a simultaneous combination with other local, active substances and / or interactive wound dressings is in principle unnecessary. Contact with non-precious metals should be avoided. The wound can be covered with modern wound dressings after applying ActiMaris® Sensitiv wound irrigation solution.

On the other hand, a good compatibility with modern wound dressings such as silver dressings has been shown in clinical trials. Likewise, there is ample clinical evidence of good tolerability when combined with inert, non-active bandages and wound dressings such as compresses, gauze, pads, sponges, gels, hydro fibers, alginates, hydrocolloids etc. The concomitant use of ActiMaris® Sensitiv wound irrigation solution with the ActiMaris® wound gel is possible and useful.

### 10. General safety instructions

ActiMaris® Sensitiv wound irrigation solution is suitable only for external wounds on skin or mucosa. Do not use for infusion or injection! Do not swallow!

Use only untouched and undamaged containers. Keep out of reach of children!

It is possible for colored fabrics to fade after contact with ActiMaris® products.

### 11. Composition and technical information

Sea salt, water, sodium oxychlorite.

Clear, colorless, alkaline and hypertonic sea salt solution with characteristic smell; contains 1.2% sea salt (Sal Maris), 0.04% NaOCl and HOCl (0,0044%).

### 12. Storage specification, duration, expiry date

ActiMaris® Sensitiv wound irrigation solution has to be stored at room temperature between +5° and +25°C. Protect the bottle from direct sunlight.

Shelf life is 30 months. Expiry date is on the packaging.

The period of use is 3 months after opening the bottle. Close carefully after use.

#### ActiMaris AG Manufacturer:

ActiMaris AG  
Sandgrube 29  
9050 Appenzell  
info@actimaris.com  
Tel.: +41 71 505 75 25

- LOT/REF Lot number
- Order number
- Follow instructions for use
- Use by: year and month
- Storage (temperature)
- Protect from direct sunlight
- Manufacturer

Date of information: 12/2018 - V6.1

Medical product: Klasse II b, CE 1250